

Set-up Sheet



Patient: _____ Date: _____

Notes: _____

Instructions for transferring setup locations:

- When indexing to the Universal Couchtop use the indexing marks along the side of the couchtop for indexing Lok-Bars and the indexing marks along the top of the couchtop for indexing Body Pro-Lok devices. When transferring a setup from the Body Pro-Lok platform to the Universal Couchtop use the indexing marks along the top of the couch for all devices.
- When transferring a setup from the Body Pro-Lok platform to the Universal Couchtop, bridges must be set one index mark **higher** (5mm) than on the Body Pro-Lok platform.
- When transferring a setup from the Body Pro-Lok platform to the Body Pro-Lok Rails-Only System, bridges must be set five index marks **lower** (25mm) than on the Body Pro-Lok platform.
- When transferring a setup from the Universal Couchtop to the Rails-Only system, bridges must be set six marks **lower** (30mm) than on the Universal Couchtop.



Universal Couchtop



Rails-Only System


If utilizing the Platform or Rails-Only System start here, if indexing directly to the Universal Couchtop skip ahead to #3.

1. Lok-Bars™ for indexing the Platform or Rails-Only System are indexed to table at: _____

2. Rails-Only System with Exact™ Indexing - Check here if utilized

CT Couchtop Thickness: _____

3. Lateral Platform Position

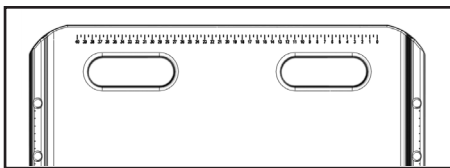

A. Head End Indexed at: _____

B. Foot End Indexed at: _____

4. Additional Two-Pin indexable accessory: _____

Indexed at: _____

5. Additional Two-Pin indexable accessory: _____


Indexed at: _____

6. Additional Two-Pin indexable accessory: _____

Indexed at: _____

7. Forehead Brace - Check here if utilized

Bridge Type Used (Check One) - Type 1 Type 1B Type 2 Type 3 Indexed at: _____

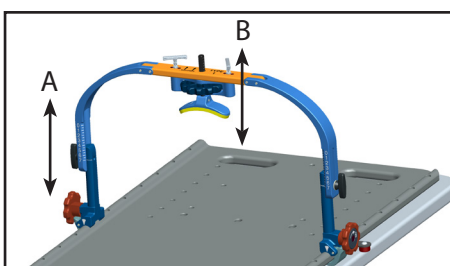


A. Bridge Height: _____

B. Forehead Brace Height: _____

Set-up Sheet




8. Shoulder Restraint Bridge - Check here if utilized Indexed at: _____

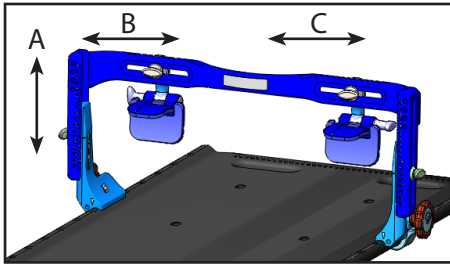

A. Bridge Height: _____
 B. Shoulder Restraint Width (right): _____
 i. Shoulder Cup Tilt* (right): _____
 C. Shoulder Restraint Width (left): _____
 ii. Shoulder Cup Tilt* (left): _____


* Shoulder Cup Tilt not available on all models

9. Respiratory Plate - Check here if utilized
 Bridge Type Used (Check One) - Type 1 Type 1B Type 2 Type 3 Indexed at: _____

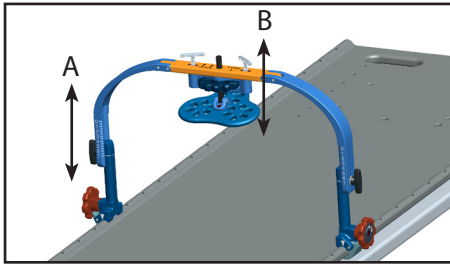

A. Bridge Height: _____
 B. Respiratory Plate Height: _____


10. Respiratory Belt - Check here if utilized
 Indexed at: _____

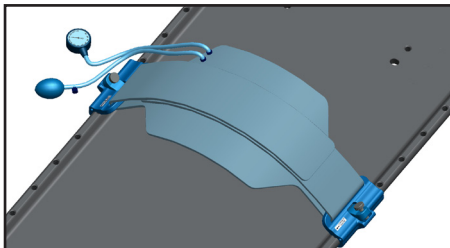

Left Velcro Strap Indexed at: _____ - _____
 Right Velcro Strap Indexed at: _____ - _____
 Set pressure between: _____ and _____

11. Patient Hand Grips - Check here if utilized
 Right hand grip indexed at: _____ Left hand grip indexed at: _____

12. Clam-Lok™ Cushion - Check here if utilized
 Bridge Type Used (Check One) - Type 1 Type 1B Type 2 Type 3 Indexed at: _____

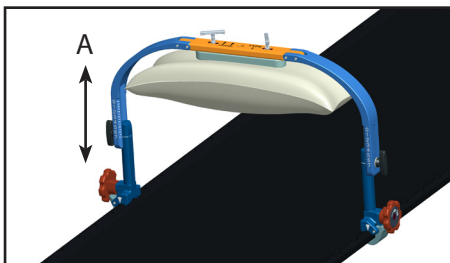

A. Bridge Height: _____

Global Offices

Orange City, Iowa, USA
 800.842.8688 | +1.712.737.8688

Rotterdam, The Netherlands
 +31 (0) 10.3036600

info@zivco.com
 www.zivco.com/ro

