IPPS™ Overlay

An indexed patient positioning system for use with GE Healthcare Global table

REFERENCE GUIDE

CIVCO

CE

INTENDED USE

Overlay is intended to provide a flat top with indexing for radiation treatment planning.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

⚠ WARNING


- Do not use if device appears damaged.
- Overlay weight: 30 lbs (14 kg). For total table load rating, including accessories, refer to your system's manual.
- Ensure device is secure prior to use.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.


ATTACHING DOCKING STATION TO CRADLE

NOTE: Tools Required: hex wrench and level (not provided).

1. Remove appropriate components from cradle.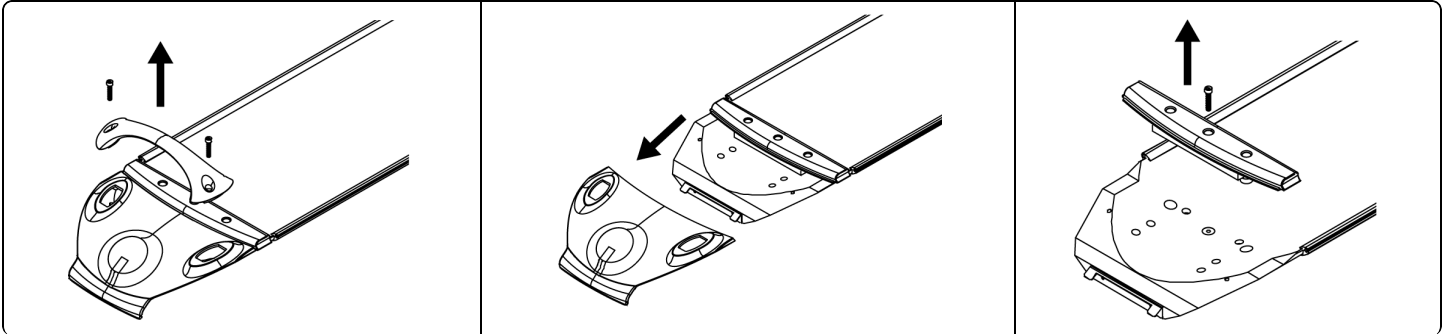


2. Attach docking station assembly to cradle.

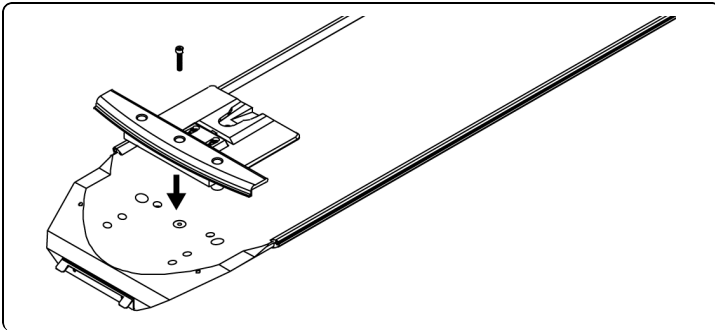ATTACHING OVERLAY

1. Loosen tongue screws with screwdriver.
2. Slide overlay into docking station, ensure tongue fits properly into accessory slot.
3. Ensure latch is locked into docking station.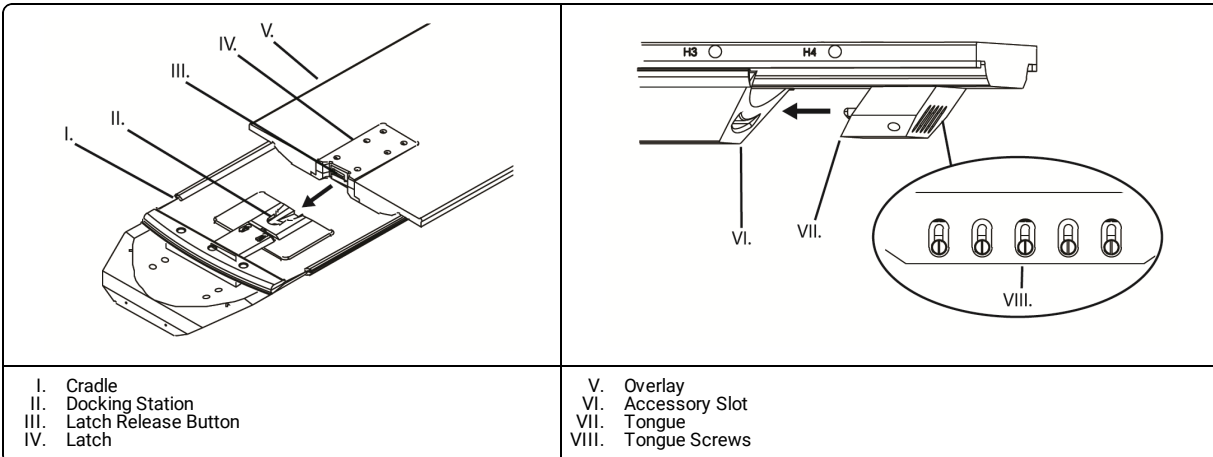


4. Tighten tongue screws to lock tongue into place.
5. Note position of docking station. Remove overlay with latch release button.
6. Tighten docking station with M5 hex wrench.

NOTE: Ensure docking station location allows for easy installation and removal of overlay.

7. Attach overlay to docking station.
8. Ensure overlay is level to within 1° laterally and longitudinally.


NOTE: If vertical height adjustment is necessary: remove latch and adjust by adding or removing shims.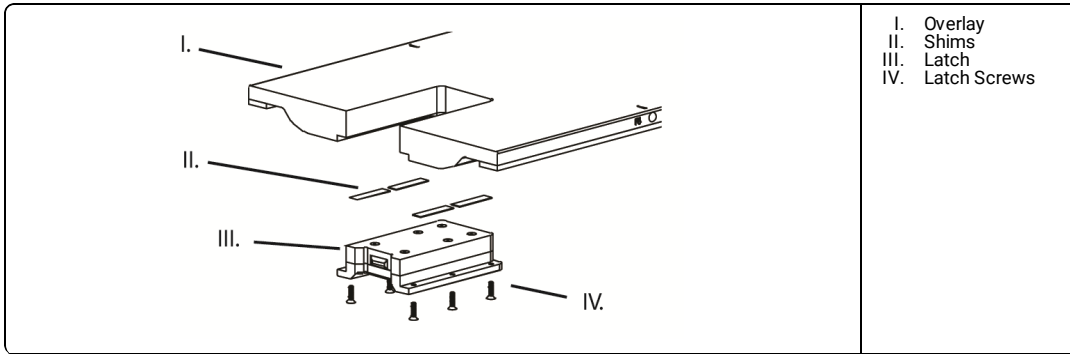

- I. Overlay
- II. Shims
- III. Latch
- IV. Latch Screws

USING OVERLAY

1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Attach positioning device onto Lok-Bar™ pins.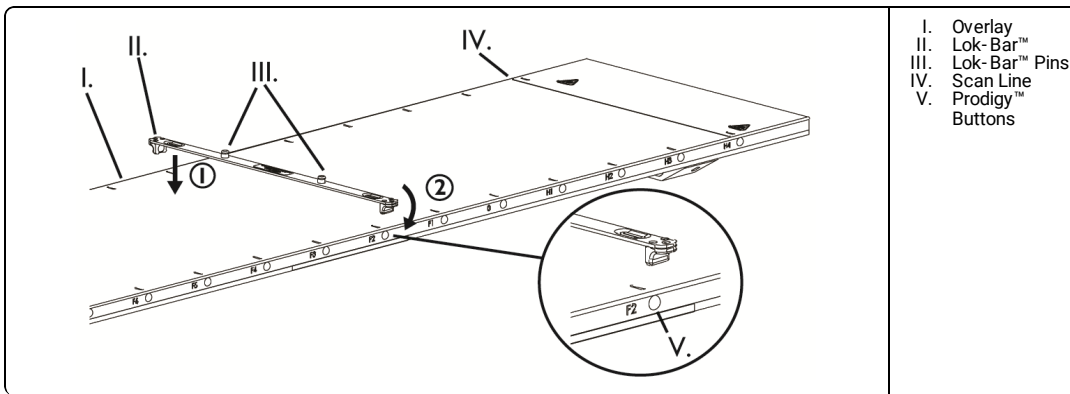


- I. Overlay
- II. Lok-Bar™
- III. Lok-Bar™ Pins
- IV. Scan Line
- V. Prodigy™ Buttons

NOTE: For optimal imaging, begin scanning after scan line.

⚠ WARNING

- Avoid pinch points when overlay is in motion by ensuring patient does not grasp overlay sides.
- Potential pinch point occurs between overlay and gantry when scanning with the table in lowest position.
- Potential pinch point occurs between overlay and table when retracting overlay.

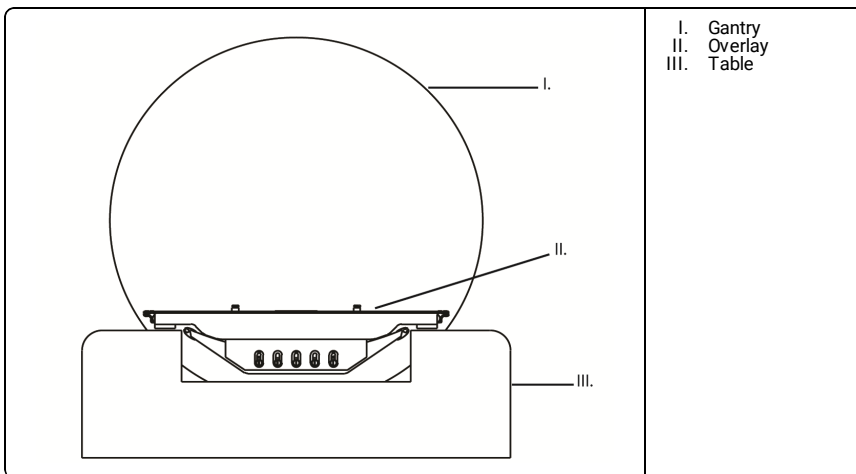- I. Gantry
- II. Overlay
- III. Table

REPROCESSING

⚠ WARNING

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Disinfect with a mild, non-abrasive cleanser.
2. Wipe dry with lint-free cloth or allow to air dry prior to reassembly and use.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2020 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. IPPS, PRODIGY, AND LOK-BAR ARE TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.