

SRS: Solstice™



Instructional Video 



Technical Data Sheets

[Solstice™ SRS Immobilization System](#)

[Solstice™ MR SRS Immobilization System](#)



Sign up for CIVCO Updates!

First Name:

Last Name:

Email:

Organization:

I have read and agree to the [privacy policy](#)

Solstice™ SRS Immobilization System

Solstice™
SRS Immobilization System

Variable pitch capability provides increased setup options in CT simulation, MR imaging (with glass fiber MR version) and corrective positioning flexibility during treatment setup

NEW! MRSeries™



Pitch at 0 (chin down)



Pitch at 5 (neutral)



Pitch at 10 (chin up)



IMRT Reinforced Thermoplastics™



ClearVision™ Open-face (with Precise Bite™)
compatible with Vision RT's Align RT camera system

Additional Features & Benefits

- Non-invasive immobilization with submillimeter accuracy for treatment of multiple lesions with a single isocenter
- Midline thermoplastic mask attachment provides increased rigidity and minimizes shrinkage to enhance patient comfort
- Highly conformal to the patient when using a customizable cushion and dedicated thermoplastic mask
- Unique thermoplastic frame provides easy to use, secure and quiet mask attachment
- Adaptable with Precise Bite™ to aid in effective and repeatable positioning



NEW! Shoulder Thermoplastic

CIVCO®
Radiotherapy

System Customization

The system is comprised of a carbon fiber head support, customizable cushion and dedicated thermoplastic mask designed for secure and simple attachment. These components create a highly customizable and comfortable head and neck immobilization system for precise treatment to improve patient outcomes.

SGRT Starter Kit

visionrt | COMPATIBLE

Kit includes: Head Support (x1), Solstice ClearVision™ Open-face Thermoplastic, Standard (x10), 20 x 35 cm Chamfered AccuForm™ Cushion (x10), Precise Bite™ (x5).

CHS03SGK	Carbon Fiber, Posifix®
CHS04SGK	Carbon Fiber, Type-S™
CHS03MRSGK	Glass Fiber MR, Posifix
CHS04MRSGK	Glass Fiber MR, Type-S

Standard Starter Kit

Kit includes: Head Support (x1), Solstice IMRT Reinforced Thermoplastic™ Style 18, Standard (x10), 20 x 35 cm Chamfered AccuForm Cushion (x10), Precise Bite (x5).

CHS03K	Carbon Fiber, Posifix
CHS04K	Carbon Fiber, Type-S
CHS03MRK	Glass Fiber MR, Posifix
CHS04MRK	Glass Fiber MR, Type-S

Individual Order Items

Head Support

CHS03	Carbon Fiber, Posifix
CHS04	Carbon Fiber, Type-S



CHS03MR	NEW! Glass Fiber MR, Posifix
CHS04MR	NEW! Glass Fiber MR, Type-S



ClearVision™ Thermoplastic

MTAPCID1832SG	Standard (shown), Qty 5
MTAPCIN1832SG	Narrow, Qty 5



visionrt | COMPATIBLE

IMRT Reinforced Thermoplastic™ Style 18

MTAPCID1832	Standard, Qty 5
MTAPCIN1832	Narrow (shown), Qty 5



Global Sales Office

Orange City, Iowa, USA
800.842.8688 | +1 712.737.8688

info@CivcoRT.com
www.CivcoRT.com

COPYRIGHT © 2020. CIVCO, AND POSIFIX ARE REGISTERED TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. CLEARVISION, IMRT REINFORCED, PRECISE BITE, SOLSTICE AND TYPE-S ARE TRADEMARKS OF CIVCO. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW. 2018P1357 REV. G


Shoulder Thermoplastic **NEW!**

MTAPSSTS	Type-S Standard White, Solid (left), Qty 5
MTAPSSPF	Posifix Standard White, Solid (right), Qty 5



AccuForm™ Cushions

MTACL2035	20 x 35 cm Chamfered, Qty 1
MTACL2035HF	20 x 35 cm Chamfered, Half Fill, Qty 1

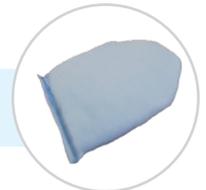

Precise Bite™

MTCBPBITE	Standard, Qty 5
MTCBPBITES	Small, Qty 5



- Vision RT is unable to comment on, nor make claims, about the immobilization rigidity or patient alignment reproducibility when using this mask;
- Compatibility is subject to users following CIVCO guidelines for forming open masks in conjunction with Surface Guided Radiotherapy Systems, and subject to users following Vision RT's guidelines for drawing Regions of Interest for use in SRS with Open Masks;
- The mask vendor is responsible for ensuring that users are in receipt of guidelines for forming masks for use in conjunction with Surface Guided Radiotherapy Systems. Vendors may change their guidelines from time to time without reviewing these changes with Vision RT. Vision RT is not liable for misuse of masks;
- Vision RT makes no comment on, and is not liable for, suitability or otherwise of masks for SRS or any other procedure;
- Purchasers are responsible for making their own choices as to the masks selected by them.

CIVCO
Radiotherapy

RESEARCH

Open Access

Intra-fraction and Inter-fraction analysis of a dedicated immobilization device for intracranial radiation treatment



Chin Loon Ong¹, Niccolò Gaj-Levra^{2*} , Luca Nicosia², Vanessa Figlia², Davide Tomasini³, Eric M. Franken¹ and Filippo Alongi^{2,4}

Abstract

Background: Immobilization devices are crucial to minimize patient positioning uncertainties in radiotherapy (RT) treatments. Accurate inter and intra-fraction motions is particularly important for intracranial and stereotactic radiation treatment which require high precision in dose delivery. Recently, a new immobilization device has been developed specifically for the radiation treatment of intracranial malignancies. To date, no data are available on the use of this device in daily clinical practice. The aim of this study is to investigate the intra and inter-fraction variations, patient comfort and radiographer confidence of the immobilization system from two distinct institutions: HagaZiekenhuis, Den Haag, Netherlands and IRCCS Ospedale Sacro Cuore Don Calabria, Negrar, Italy.

Material and method: Sixteen patients (10 diagnosed with brain metastases and 6 with primary central nervous systemic tumor) from IRCCS Ospedale Sacro Cuore Don Calabria and 17 patients (all diagnosed with brain metastases tumor) from HagaZiekenhuis were included in this study. The median target volume was 436 cc (range 3.2–1628 cc) and 4.58 cc (range 0.4–27.19 cc) for IRCCS and Haga, respectively. For patients treated in IRCCS Sacro Cuore Don Calabria, the median dose prescription was 30 Gy (range 27–60 Gy) and median number of fractions 10 (range 3–30). In Haga the median dose prescription was 21 Gy (range 8–21 Gy) and the median number of fraction was 1 (range 1–3). The immobilization device was assembled during CT simulation. A short interview to the patient regarding the device's comfort level was conducted at the end of the simulation procedure. Additionally, simulation setup time and radiographer (RTT) procedures (i.e. mask preparation) were evaluated. Prior to radiation treatment delivery, an automatic rigid match on the cranial bones between cone beam computed tomography (CBCT) and planning-CT was performed. A couch shift was performed subsequently. An extra post-treatment CBCT was acquire after the treatment delivery. This post-treatment CBCT was matched with pre-treatment CBCT to identify any possible intra-fraction motion. All online matches were validated by experienced radiation oncologist or RTT. A total of 126 CBCT's were analyzed offline by radiation oncologist/medical physicist. The data of the pre-treatment CBCT match was used to quantify inter-fraction motion. The post-treatment CBCT was matched with pre-treatment CBCT to identify any possible intra-fraction motion.

(Continued on next page)

* Correspondence: niccolo.gajlevra@sacrocuore.it

²Advanced Radiation Oncology Department, IRCCS Sacro Cuore Don Calabria Hospital, Via Don A.Sempreboni 5, 37024, Negrar di Valpolicella, Verona, Italy
Full list of author information is available at the end of the article



© The Author(s). 2020 **Open Access** This article is licensed under a Creative Commons Attribution 4.0 International License, which permits use, sharing, adaptation, distribution and reproduction in any medium or format, as long as you give appropriate credit to the original author(s) and the source, provide a link to the Creative Commons licence, and indicate if changes were made. The images or other third party material in this article are included in the article's Creative Commons licence, unless indicated otherwise in a credit line to the material. If material is not included in the article's Creative Commons licence and your intended use is not permitted by statutory regulation or exceeds the permitted use, you will need to obtain permission directly from the copyright holder. To view a copy of this licence, visit <http://creativecommons.org/licenses/by/4.0/>. The Creative Commons Public Domain Dedication waiver (<http://creativecommons.org/publicdomain/zero/1.0/>) applies to the data made available in this article, unless otherwise stated in a credit line to the data.

(Continued from previous page)

Results: During the molding of the mask, all patients responded positive to the comfort. Median time required by the RTTs to assemble the immobilization system was 9 min (range 6–12 min). In terms of comfort, all patients reported a good-to high level of satisfaction. The RTTs also respond positively towards the use of the locking mechanism and clips. Results of positioning uncertainties were comparable between the two institutes. The mean inter-fraction motion for all translational and rotational directions were < 2 mm (SD < 4 mm) and $< 0.5^\circ$ (SD $< 1.5^\circ$), respectively, while the mean intra-fraction motions were < 0.4 mm (SD < 0.6 mm) and 0.3° (SD $< 0.5^\circ$).

Conclusions: This study demonstrates the efficacy and feasibility of the immobilization device in the intracranial radiation treatment. Both patient comfort and preparation time by RTTs are considered adequate. In combination with online daily imaging procedure, this device can achieve submillimeter accuracy required for intracranial and stereotactic treatments.

Keywords: Radiotherapy, Immobilization, Inter-fraction, Intra-fraction, Brain

Introduction

Accuracy in radiation treatment is considered one of the most relevant issues in modern radiotherapy (RT) [1]. This concept included two distinct aspects, the delivery of high radiation doses to the tumor (e.g. stereotactic cranial and extracranial RT and hypofractionation) and the decrease in normal tissue irradiation. To achieve this balance, several aspects have to be considered: i) the precise definition and verification of the oncological target, supported by radiological and metabolic images (Image guided radiotherapy therapy – IGRT), ii) the use of modern radiotherapy delivery techniques (Intensity Modulated Radiotherapy – IMRT and volumetric modulated arc therapy – VMAT) and finally, during radiation delivery, iii) limiting the inter- and intra-fraction motion with suitable immobilization devices (and, if available, real-time monitoring system such surface guided systems).

All immobilization systems designed for radiation treatment should meet several conditions. The capability of reducing positioning errors and the limiting patient movements alone are not considered sufficient. Good comfort for the patient and short time for the construction of the device by radiographers (RTTs) are also important. One of the most relevant aspects recently explored in the literature was the role of immobilization devices, focusing on intracranial treatments and in particular for stereotactic radiotherapy [2–4]. Most commercially available immobilization devices have been evaluated [5]. Recently, a new immobilization device dedicated to the treatment of intracranial disease, including in particular high precision in stereotactic radiotherapy treatment and patient comfort in conventional intracranial treatments. To date, there are not data about its clinical application and the aim of this study is to investigate the setup accuracy of this device. Furthermore, we also collected user experience for this device based on both patients' and RTTs' perspectives.

Material and methods

In this observational study, we investigated the inter- and intra-fraction variations of the Solstice™ SRS

Immobilization System, CIVCO Radiotherapy (Kalona, USA) device for precise intracranial radiation treatments. Moreover, we evaluated patient comfort and the time required for preparation by RTTs for molding the system. Data collected from 2 different institutes, Haga-Ziekenhuis (Haga), Den Haag, Netherlands and IRCCS Ospedale Sacro Cuore Don Calabria, Negrar, Italy, were analyzed. From both institutions, the inclusion criteria were: a) age > 18 years, (b) diagnosis of oncological brain disease eligible to RT, (c) informed consent. Exclusion criteria were: (a) patients not eligible to RT, (b) claustrophobic patients. Focusing on dose prescription, IRCCS Ospedale Sacro Cuore Don Calabria included patients eligible to standard fractionation or moderate hypofractionation, while HagaZiekenhuis enrolled only patients receiving hypofractionated treatments.

Immobilization device and CT simulation

The Solstice system comprised of a carbon fiber head support, customizable accuform cushion and thermoplastic mask (Fig. 1). The head support allows manual pitch setup errors correction by rotating the screw located at the back of the system. Two RTTs were responsible for the construction of the thermoplastic mask and customizable cushion for each patient. The total set up time was calculated, including all the procedures required: from the recline patient position on the CT simulation couch to preparation of the cushion, molding and cooling down of the mask, and finally the acquisition of CT images. Three distinct landmarks were positioned to the mask (1 frontal and 2 laterals). CT simulation was performed without contrast media and the scan length included the whole brain (with a special resolution of 0.30 mm from both institutions). Slice thickness varied between 1 to 3 mm, depending on different internal treatment protocols.

At the end of each procedure RTTs reported in a specific form, any limitation or problem recorded during the procedure. Specifically, the following procedures have been evaluated: 1) pitch locking level (ease of use,

ease of locking indentation and stability of lock), 2) mask clips (ease of use). RTTs had the possibility to mark four different options: poor, fair, good, excellent. After CT simulation, a radiation oncologist interviewed the patient in order to collect information about comfort: 1) Did you have a good comfortable head position to the accuform cushion? 2) Did you have a good comfortable with the mask? 3) Did you feel an extreme mask pressure on your face? 4) Do you feel that the simulation procedure was too long? 5) Extra feedback(s) from the patients. The patients could answer each question by rating it as poor, fair, good or excellent. This information was recorded in the form, with the aim to compare patient comfort and any possible limitation detected by RTTs during the mask preparation. Finally, radiation oncologist conversed with RTTs, in order to record any potential problems observed during the mask procedure. A dedicated homemade questionnaire was created and used from both centers.

Target volume definition

In IRCCS Ospedale Sacro Cuore, the target volume definition was different according to histology and radiation dose prescription. In high grade gliomas, the gross tumor volume (GTV) was defined as the surgical cavity or residual disease or macroscopic disease detected on T1 sequence on magnetic resonance images (MRI) with contrast. The clinical target volume (CTV) was obtained by adding a 15 mm isotropic margin from the GTV. The planning target volume (PTV) was achieved adding a 5 mm margin to the CTV. For whole brain radiotherapy in multiple brain metastases, CTV was the entire brain and a 5 mm isotropic margin was used to create the PTV. For stereotactic radiation treatment, GTV was defined as the macroscopic disease detected on T1 contrast sequence on MRI. The PTV margin use for IRCCS for stereotactic treatment was 2 mm. In Haga, the GTV was similarly defined using a fusion between T1 contrast sequence on MRI and CT simulation. No CTV margin was used for hypofractionated treatment and an isotropic margin of 1 mm was used to create the PTV. For both institutions, the contouring software provided a dedicated rigid fusion tool between MR images and CT simulation. The rigid fusion was allowed according to the cranial immobility. Radiation oncologists had a high level of confidence and experience to evaluate the fusion quality between MR and CT images. In stereotactic radiation treatments, PTV margins used by IRCCS and Haga were justified by DEGRO guidelines [3].

In IRCCS, the dose prescriptions as followed: high grade glioma 60 Gy in 30 fractions (2 Gy per fraction), multiple brain metastases 30 Gy in 10 fractions (3 Gy per fraction), while in stereotactic intracranial treatment, dose prescription was 27 Gy in 3 fractions (9 Gy per

fraction). In Haga, all stereotactic patients were treated with 16–21 Gy in 1 fraction or 18 Gy – 25.5 Gy in 3 fractions (6–8.5 Gy), depends on the PTV volume, its proximity to the organs at risk and histology [6].

Positioning workflow

In IRCCS Ospedale Sacro Cuore Don Calabria, all patients were treated with a TrueBeam™ (Varian Medical System) v. 2.0 Perfect Pitch configuration, due to the high precision in the definition of movement variation in all directions (6D positioning system). In Haga, all patients were treated with using Elekta Synergy Agility linear accelerator in combination with the pitch-rotational functionality available with the Solstice system. The online imaging procedures were slightly different between the 2 institutes.

In IRCCS, a single CBCT was acquired before each radiation treatment. A rigid match between simulation CT and CBCT images was performed automatically by the software, using cranial bones as focus point and validated by a radiation oncologist and RTTs. The setup error tolerance was difference in conventional or SRS treatments. In the conventional fractionation shift tolerance in translational and rotational inter-fraction motions were ≤ 7 mm and 3° , while ≤ 2 mm and 2° was applied in SRS treatments. In all cases a 6D correction was executed. If the tolerance was exceeded, the patient was repositioned and the entire procedure was repeated. At the end of the session a post-treatment CBCT was performed with the aim to identify patient movements during the treatment delivery (intra-fraction motions).

In Haga, at least 2 CBCT's were acquired before the radiation treatment. The first CBCT were acquired and a rigid registration was performed based on cranial bone. Subsequently, setup errors in all translational directions were corrected. If a pitch rotational error of $> 1^\circ$ was detected, this would be manually corrected with the Solstice system. The roll and yaw rotational errors were automatically translated into translational corrections in XVI software during the match. The accuracy of this adjustment was verified with a second CBCT. For each radiation delivery, online CBCTs were validated by experienced radiation oncologist or RTT. If all translational and rotational setup errors were smaller than 1 mm/ 3° , treatment fields will be delivered. The data collection was performed off-line by a medical physicist after the end of each session.

Patients

Between January 2018 and August 2019, a total number of 126 pre (63) and post treatment (63) CBCTs were analyzed, from 16 and 17 patients with a diagnosis of intracranial oncological disease tumor treated at IRCCS and Haga, respectively. In IRCCS, ten (62.5%) patients had a

diagnosis of brain metastases, while 6 (37.5%) reported a primary central nervous systemic tumor. All patient treated in Haga were diagnosed with brain metastases tumor. Median target volume was 436 cc (range 3.2–1628 cc) and 4.58 cc (range 0.4–27.19 cc) for IRCCS and Haga, respectively.

The median dose prescription was 30 Gy (range 27–60 Gy), and 21 Gy (range 8 Gy – 21 Gy) for IRCCS and Haga, respectively. The median number of fractions was 10 (range 3–30), and 1 (range 1–3) for IRCCS Calabria and Haga, respectively. The details of patient characteristics and dose prescriptions are listed in Table 1.

Set-up error, inter and intra-fraction data collection

The inter-fraction variability was obtained by matching the first CBCT with planning-CT. The same radiation oncologist reviewed off-line the images to confirm the quality of the match, focusing on bone structures and air cavities matching. If the match was suboptimal, a second radiation oncologist will perform a double check.

A standardized off-line procedure has been used to collect data, with the support of ARIA® version 15.1 – Varian™ (IRCCS) and Elekta XVI version 5.0 (Haga). Match values of all three translational axis (x = lateral, y = longitudinal, z = vertical) and three rotational axes (roll, pitch and yaw) from the very first CBCT were recorded in order to establish the daily pre-treatment setup errors (inter-fraction variation). Similar procedure was used to match the post-treatment CBCT to the CBCT acquired right before treatment delivery (intra-fraction variation). Additionally, in order to quantify the deviations in 3D space, a “displacement vector” (D vector) was defined from 3 axes data.

Results

The feedbacks from all patients and RTTs are presented in Table 2. More than 80% of the patients graded the comfort of the immobilization devices as “Good” and “Excellent”. The median setup time for patients was 9 min (range 6–12 min). And this time is positively experienced by 98% of the patients. RTT did not reported any critical technical issue in the molding and fixation of the

mask during all the procedures from simulation to treatment delivery. Most of them reviewed the pitch locking mechanism, stability of the lock and the mask clip very positively. We also did not observe any specific problem with the coach shift after CBCT corrections.

Inter-fraction variability

Mean and Standard deviation (SD) of the inter-fraction motion for hypofractionated patients from both institutes for all translational and rotational directions are presented in Fig. 2. The mean inter-fraction motion for all translational and rotational directions were < 2 mm (SD < 4 mm) and < 0.5° (SD < 1.5°), respectively. The mean 3D-vector displacement of the inter-fraction variability for IRCCS and Haga were 0.23 and 1.18 mm, respectively. The inter-fraction motion for patients treated with normal fractionated schemes were presented in Fig. 4. The mean inter-fraction motion for all translational and rotational directions were < 2 mm (SD < 4 mm) and < 1° (SD < 2.5°), respectively.

Intra-fraction variability

The intra-fraction mean values were obtained by the match between pre-treatment CBCT and post-treatment CBCT. The mean and SD of the inter-fraction motion for both institutes for all translational and rotational directions are presented in Fig. 3. The mean intra-fraction motions for all translational and rotational directions were < 0.4 mm (SD < 0.6 mm) and 0.3° (SD < 0.5°), respectively. The mean 3D-vector displacement of the intra-fraction variability for IRCCS and Haga were 0.13 and 0.26 mm, respectively. The intra-fraction motion for patients treated with normal fractionated schemes were presented in Fig. 4. The mean inter-fraction motion for all translational and rotational directions were < 1.8 mm (SD < 2 mm) and < 0.3° (SD < 0.5°), respectively.

The systematic and random errors for hypofractionated patients were calculated and presented in Table 3. These errors were calculated separately for inter- and intra-fraction motions.

Table 1 Patient characteristics and dose prescriptions at IRCCS Sacro Cuore Don Calabria and HagaZiekenhuis

	IRCCS Ospedale Sacro Cuore Don Calabria	HagaZiekenhuis
Number of patients (%)	16 (100%)	17 (100%)
Male (%) and female (%)	7 (43.8%) and 9 (56.2%)	10 (58.8%) and 7 (41.2%)
Brain metastases cases (%)	10 (62.5%)	17 (100%)
Primary CNS tumor cases (%)	6 (37.5%)	0
Median target volume (cc) (range)	436 (3.2–1628 cc)	4.58 (0.40–27.19 cc)
Median dose prescription (range)	30 Gy (27–60 Gy)	21 Gy (8–21 Gy)
Median Number of fraction (range)	10 (3–30)	1 (1–3)

CNS central nervous system

Table 2 Patients and Radiographer feedback on the immobilization device

Feedback	Poor (%)	Fair (%)	Good (%)	Excellent (%)
Patient's feedback				
Did you feel comfortable with the accuform cushion?	0	18	31	51
Did you feel comfortable with the mask?	0	0	38	62
Did you feel extreme pressure of the mask on your face?	0	2	40	58
Do you feel that the simulation procedure was too long?	0	2	8	90
RTT's feedback				
Ease of use of the pitch locking mechanism	0	0	30	70
Ease of locking identification and stability of the lock	0	0	20	80
Ease of use of the mask clips	0	0	0	100

Discussion

Over the years, several non-invasive stereotactic immobilization system [4–9] and bite blocks [10–12] were introduced. Recently, the Solstice™ immobilization device has been developed and up to date, there are still no data about its clinical application. In this dual centers study, we analyzed the intra and inter-fraction accuracy of the Solstice immobilization system during conventional and stereotactic treatment.

At first, we evaluated patient tolerability and radiographer comfort in the use of this immobilization device. The results of our experience showed that RTTs felt confident with the mask. We also observed a fast learning curve and a progressive decrease in time for mask preparation. In terms of comfort, all patients reported a good-to high level of satisfaction.

The results of inter and intra-fraction variations of both institutes were comparable. For translational and rotational directions, the mean inter-fraction motion was < 1 mm (SD < 4 mm) and < 0.5° (SD < 1.5°), respectively. Daily IGRT procedure, using CBCT, is able to

detect patient positioning errors. Hence, these errors are usually corrected before treatment delivery. In terms of treatment accuracy, intra-fraction motions play a more important role. In both institutes, the mean intra-fraction motions for all translational and rotational direction were < 0.2 mm (SD < 0.6 mm) and 0.5° (SD < 0.6°), respectively. This is within the 1 mm PTV margin commonly used for stereotactic radiation treatment [13].

Our results are comparable to current literature on non-invasive stereotactic immobilization systems, despite different measuring and statistical methods were applied [14–21]. One strength point of our approach was the comparison between pre- and post-treatment CBCT. As supported by the literature [14], the use of 6D couch allowed a high precision in detecting positioning variations. In particular, Guckenberger et al. demonstrated that the integration of image guidance significantly affects reducing set-up error from 3.9 ± 1.7 mm to 0.9 ± 0.6 mm [15]. In our experience, the set-up errors were 0.23 mm at IRCCS Ospedale Sacro Cuore Don Calabria and 1.18 mm (3D vector) HagaZiekenhuis respectively,

Table 3 Systematic and random errors for hypofractionated patients

		Systematic errors (mm/°)		Random errors (mm/°)	
		Haga	IRCCS	Haga	IRCCS
Inter-fraction	Longitudinal	0.98	1.59	1.53	1.67
	Lateral	0.61	0.63	1.83	2.10
	Vertical	0.22	0.27	1.23	1.31
	Roll	0.40	0.07	0.77	0.92
	Pitch	0.10	0.19	0.74	1.23
	Yaw	0.24	0.05	1.11	0.96
Intra-fraction	Longitudinal	0.05	0.01	0.23	0.50
	Lateral	0.17	0.10	0.28	0.41
	Vertical	0.19	0.26	0.35	0.17
	Roll	0.01	0.04	0.36	0.19
	Pitch	0.20	0.04	0.24	0.34
	Yaw	0.09	0.06	0.34	0.23

confirming the CBCT accuracy for the isocenter identification.

Analyzing intra-fraction motion, the current literature reported heterogeneous results, due to the use of different immobilization systems. Intra-fraction 3D vector varied between 0.5 mm to 3.9 mm [15–19]. Our report shows superior intra-fraction 3D vector displacement of 0.13 and 0.26 mm for IRCCS and Haga, respectively. For rotational errors, Babic et al. reported the mean rotational errors between -0.20° and 0.33° for a variety of immobilization devices [5]. Additionally Nielsen et al. explored the use of 6D Hepadox, intracranial and extracranial radiation treatments. In intracranial treatments, they observed a mean residual rotational setup error of 0.06° (SD 0.3°) [22]. This is similar to our reported mean intra-fraction variabilities of between -0.2° and 0.47° [5].

The inter-fraction positioning based on stereotactic coordinates is heterogeneous. Accuracy and reproducibility data about patient repositioning varied according to the immobilization system used (with or without bite block). Isocenter deviation varied between $0.5 \text{ mm} \pm 0.7 \text{ mm}$ in the experience published by Minniti et al. [6] and 3.7 mm when mask immobilization was used alone [20]. Nevertheless, a more recent article published by Ramakrishna et al. [21] did not record any significant intra-fraction variation in patients treated with radiosurgery using a frame-based versus a frameless image-guided system. Analyzing our data, the use of frameless radiotherapy supported by CBCT was associated with comparable results published in these literatures. We did observe an outlier with a deviation of 10.9 mm in longitudinal direction in one single fraction. This value was observed at the last radiation dose delivery in a patient with a loose weight during the radiation treatment, while the treatment mask consistency was preserved.

The limitations of this study are the limited number of patients enrolled, the heterogeneous population selected, different dose prescriptions and radiation treatment margins. For all patients enrolled in this study, the immobilization device and IGRT procedure remain the same within one institutes. In IRCCS, the inter-fraction uncertainty is larger in patients treated with normal fractionated treatment than hypofractionated treatment. With a longer overall treatment time for normal fractionated treatment, there might be more positioning uncertainties caused by any possible anatomical changes or other time factors. This could lead to larger inter-fraction motion as patients might be positioned slightly differently in the beginning of each treatment. However, once the mask was placed, the patients were firmly immobilized, resulting in comparable intra-fraction uncertainty between the normal fractionated and hypofractionated treatment. And although the IGRT procedures were slightly different between the 2 institutes, the

resulting inter- and intrafraction uncertainties were comparable as shown in Fig. 2 and 3. Due to limited normal fractionated patients enrolled in this study, the systematic and random errors were only calculated for patients treated with hypofractionated schemes. These errors were calculated separately for inter- and intra-fraction motions. A combined systematic and random errors were not calculated as the inter-fraction motions were corrected before the treatment. For intra-fraction motion, both systematic errors and random errors were $< 0.3 \text{ mm}/^\circ$ and $\leq 0.5 \text{ mm}/^\circ$, respectively.

We demonstrated the potential application of Solstice™ in several clinical scenarios, without any negative impact in intra and inter-fraction values. Additionally, the comparable data resulting from two independent institutions, using two different IGRT procedures, can also further support the reliability and consistency of the performance of the mask, which can be used under different scenarios. Finally, despite limited amount of patients included in this study, the total number of CBCT evaluated is acceptable to support our hypothesis.

Conclusions

This report showed that Solstice™ SRS Immobilization System, CIVCO Radiotherapy is feasible and efficient for treating patients with intra-fraction lesion. Additional good feedback has been reported by both patients and radiographers. In combination with daily CBCT, the Solstice system could achieve submillimeter positioning accuracy, which is required for high precision stereotactic treatment.

Abbreviations

RT: radiotherapy; SD: standard deviation; IGRT: imaged guided radiotherapy; IMRT: intensity modulated radiotherapy; VMAT: volumetric modulated arc therapy; RTT: radiographers; CBCT: cone beam computed tomography

Acknowledgements

Not applicable.

Authors' contributions

OLC, NGL, FA conceived the study, analyzed and interpreted data and wrote manuscript; LN, VF: collected data, review and revision of manuscript. DT: collected data. EMF: analyzed and interpreted data, review and revision of manuscript. All authors approved the final version of the manuscript.

Funding

Not applicable.

Availability of data and materials

The patient information may be shared under 'IRCCS Sacro cuore – Don Calabria' hospital and HagaZiekenhuis, Den Haag, Netherlands. IRB approval of amendment on a case by case base. The Solstice™ SRS Immobilization System is proprietary CIVCO Radiotherapy due to patient protection.

Ethics approval and consent to participate

All patients signed an informed consent for the treatment.

Consent for publication

Not applicable.

Competing interests

The authors declare that they have no competing interests.

Author details

¹Department of Radiotherapy, HagaZiekenhuis, Den Haag, The Netherlands. ²Advanced Radiation Oncology Department, IRCCS Sacro Cuore Don Calabria Hospital, Via Don A.Sempreboni 5, 37024, Negrar di Valpolicella, Verona, Italy. ³Radiation Oncology Department, ASST Spedali Civili di Brescia, University of Brescia, Brescia, Italy. ⁴University of Brescia, Brescia, Italy.

Received: 26 April 2020 Accepted: 10 August 2020

Published online: 18 August 2020

References

- Bhide SA, Nutting CM. Recent advances in radiotherapy. *BMC Med.* 2010;8:25. <https://doi.org/10.1186/1741-7015-8-25>.
- Alongi F, Fiorentino A, Mancosu P, Navarria P, Gajj Levrà N, Mazzola R, et al. Stereotactic radiosurgery for intracranial metastases: linac-based and gamma-dedicated unit approach. *Expert Rev Anticancer Ther.* 2016;16:731–40. <https://doi.org/10.1080/14737140.2016.1190648>.
- Kocher M, Wittig A, Piroth MD, Treuer H, Seegenschmiedt H, Ruge M, et al. Stereotactic radiosurgery for treatment of brain metastases. A report of the DEGRO working group on stereotactic radiotherapy. *Strahlenther Onkol.* 2014;190:521–32. <https://doi.org/10.1007/s00066-014-0648-7>.
- Pilipuf MN, Goble JC, Kassell NF. A noninvasive thermoplastic head immobilization system. Technical note *J Neurosurg.* 1995;82:1082–5. <https://doi.org/10.3171/jns.1995.82.6.1082>.
- Babic S, Lee Y, Ruschin M, Lochray F, Lightstone A, Atenafu E, et al. To frame or not to frame? Cone-beam CT-based analysis of head immobilization devices specific to linac-based stereotactic radiosurgery and radiotherapy. *J Appl Clin Med Phys.* 2018;19:111–20. <https://doi.org/10.1002/acm2.12251>.
- Minniti G, Scaringi C, Clarke E, Valeriani M, Osti M, Enrici RM. Frameless linac-based stereotactic radiosurgery (SRS) for brain metastases: analysis of patient repositioning using a mask fixation system and clinical endpoints. *Radiat Oncol.* 2011;6:158. <https://doi.org/10.1186/1748-717X-6-158>.
- Willner J, Flentje M, Bratengeier K. CT simulation in stereotactic brain radiotherapy—analysis of isocenter reproducibility with mask fixation. *Radiother Oncol.* 1997;45:83–8. [https://doi.org/10.1016/s0167-8140\(97\)00135-7](https://doi.org/10.1016/s0167-8140(97)00135-7).
- Karger CP, Jakel O, Debus J, Kuhn S, Hartmann GH. Three-dimensional accuracy and inter-fractional reproducibility of patient fixation and positioning using a stereotactic head mask system. *Int J Radiat Oncol Biol Phys.* 2001;49:1493–504. [https://doi.org/10.1016/s0360-3016\(00\)01562-5](https://doi.org/10.1016/s0360-3016(00)01562-5).
- Alheit H, Dornfeld S, Dawel M, Alheit M, Henzel B, Steckler K, et al. Patient position reproducibility in fractionated stereotactically guided conformal radiotherapy using the BrainLab mask system. *Strahlenther Onkol.* 2001;177:264–8. <https://doi.org/10.1007/pl00002407>.
- Wong VY, Tung SY, Leung TW, Ho KH. CT verification of isocentre relocatability using stereotactic mask fixation system. *Clin Oncol (R Coll Radiol).* 2003;15:280–7. [https://doi.org/10.1016/s0936-6555\(03\)00091-8](https://doi.org/10.1016/s0936-6555(03)00091-8).
- Rosenthal SJ, Gall KP, Jackson M, Thornton AF Jr. A precision cranial immobilization system for conformal stereotactic fractionated radiation therapy. *Int J Radiat Oncol Biol Phys.* 1995;33:1239–45. [https://doi.org/10.1016/0360-3016\(95\)02009-8](https://doi.org/10.1016/0360-3016(95)02009-8).
- Kumar S, Burke K, Nalder C, Jarrett P, Mubata C, A'Hern R, et al. Treatment accuracy of fractionated stereotactic radiotherapy. *Radiother Oncol.* 2005;74:53–9. <https://doi.org/10.1016/j.radonc.2004.06.008>.
- Kirikpatrick JP, Wang Z, Sampson JH, McSherry F, Herndon JE, Allen KJ, et al. Defining the optimal planning target volume in image-guided stereotactic radiosurgery of brain metastases: results of a randomized trial. *Int J Radiat Oncol Biol Phys.* 2015;91:100–8. <https://doi.org/10.1016/j.ijrobp.2014.09.004>.
- Robar JL, Clark BG, Schella JW, Kim CS. Analysis of patient repositioning accuracy in precision radiation therapy using automated image fusion. *Winter.* 2005;6:71–83. <https://doi.org/10.1120/jacmp.v6i1.1998>.
- Guckenberger M, Roesch J, Baier K, Sweeney RA, Flentje M. Dosimetric consequences of translational and rotational errors in frame-less image-guided radiosurgery. *Radiat Oncol.* 2012;7:63. <https://doi.org/10.1186/1748-717X-7-63>.
- Masi L, Casamassima F, Polli C, Menichelli C, Bonucci I, Cavedon C. Cone beam CT image guidance for intracranial stereotactic treatments: comparison with a frame guided set-up. *Int J Radiat Oncol Biol Phys.* 2008;71:926–3. <https://doi.org/10.1016/j.ijrobp.2008.03.006>.
- Fuss M, Salter BJ, Cheek D, Sadeghi A, Hevezi JM, Herman T. Repositioning accuracy of a commercially available thermoplastic mask system. *Radiother Oncol.* 2004;71:339–45. <https://doi.org/10.1016/j.radonc.2004.03.003>.
- Baumert BG, Egli P, Studer S, Dehing C, Davis JB. Repositioning accuracy of fractionated stereotactic irradiation: assessment of isocentre alignment for different dental fixations by using sequential CT scanning. *Radiother Oncol.* 2005;74:61–6. <https://doi.org/10.1016/j.radonc.2004.08.002>.
- Gevaert T, Verellen D, Engels B, Depuydt T, Heuninckx K, Tournel K, et al. Clinical evaluation of a robotic 6-degree of freedom treatment couch for frameless radiosurgery. *Int J Radiat Oncol Biol Phys.* 2012;83:467–74. <https://doi.org/10.1016/j.ijrobp.2011.05.048>.
- Ramakrishna N, Rosca F, Friesen S, Tezcanli E, Zygmanski P, Hacker F. A clinical comparison of patient setup and intra-fraction motion using frame-based radiosurgery versus a frameless image-guided radiosurgery system for intracranial lesions. *Radiother Oncol.* 2010;95:109–15. <https://doi.org/10.1016/j.radonc.2009.12.030>.
- Wiggeraad R, Verbeek-de KA, Kal HB, Taphoorn M, Vissers T, Struikmans H. Dose-effect relation in stereotactic radiotherapy for brain metastases. A systematic review. *Radiother Oncol.* 2011;98(3):292–7.
- Nielsen M, Hansen CR, Brink C, Bertelsen AS, Kristiansen C, Stefan SJ, et al. Efficient and accurate stereotactic radiotherapy using flattening filter free beams and HexaPOD robotic tables. *J Radiosurg SBRT.* 2016;4:153–61.

Publisher's Note

Springer Nature remains neutral with regard to jurisdictional claims in published maps and institutional affiliations.

Ready to submit your research? Choose BMC and benefit from:

- fast, convenient online submission
- thorough peer review by experienced researchers in your field
- rapid publication on acceptance
- support for research data, including large and complex data types
- gold Open Access which fosters wider collaboration and increased citations
- maximum visibility for your research: over 100M website views per year

At BMC, research is always in progress.

Learn more biomedcentral.com/submissions



Clinical verification of the pitch functionality and setup accuracy of the Solstice SRS immobilization system

Introduction

Reliable patient immobilization is one of the most crucial part in treatment of frameless stereotactic radiosurgery (SRS) of brain metastases on a linear accelerator. Various commercial products offer promising rigid fixation, but most of these system are solely used for immobilization as any setup errors can only be corrected by the treatment couch. And if any pitch-rotational setup errors were detected, it can only be corrected using a 6D-couch, which is not commonly available.

The purpose of this study was to investigate the stability of the new CIVCO Solstice SRS immobilization system and the accuracy of its manually adjustable pitch-rotational functionality.

Method

The Solstice mask comprised of a carbon fiber head support, customizable cushion and thermoplastic mask (Fig. 1). Pitch setup errors can be corrected by rotating the screw at the back of the head support with a rotation of 0.5° per turn. Initially, 12 palliative cranial patients receiving 1 – 10 RT fractions were treated using this system, after which it was also applied to 16 SRS patients receiving 1 – 3 fractions. For all patients, 2 CBCT's were acquired prior to the treatment delivery and the detected translational shifts were corrected with treatment couch. An extra verification post-CBCT was acquired after the treatment. This CBCT was used to determine the overall setup accuracy of the system. All CBCT's were automatically registered to the planning CT based on bony anatomy. For SRS patients, pitch setup errors were manually adjusted if the first CBCT indicated a deviation of $\geq 1^\circ$. The accuracy of the pitch correction were evaluated using match results of the second CBCT's.

Conclusion

The Solstice system provides sub-millimetric positioning accuracy that is suitable for high precision SRS treatment delivery. It's unique design enable manual correction of the pitch-rotational setup error. This correction can be clinically importance for SRS treatment as it permits a more frequent use of single isocenter to treat patients with multiple brain metastases.

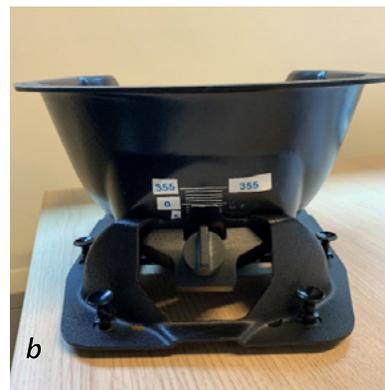
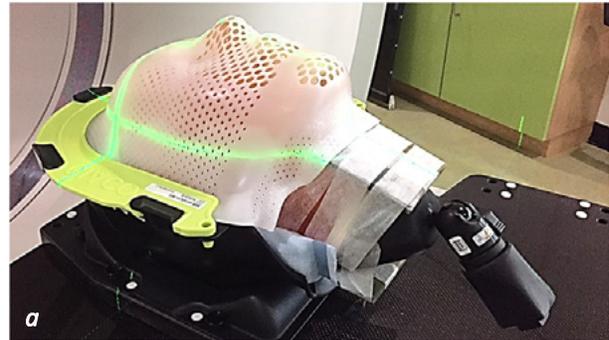


Figure 1: A phantom was fixated with the Solstice system (a). Detected pitch during the match can be corrected using the screw located at the backside of the system (b).

Results

For the first 12 palliative cranial patients, the mean-of mean (μ), systematic (Σ) and random (σ) of the translational setup uncertainties are calculated. When these values are applied to the margin formula proposed by van Herks [$M = 2.5 \Sigma + 0.7 \sigma$], the resulting setup margins for the lateral, longitudinal and vertical directions are 0.5, 0.5 and 1.0mm, respectively. For the 16 SRS patients, the means, standard deviations and maximum shifts detected with the post treatment CBCT's are shown in Table 1. In 4 fractions, pitch deviation of $>1^\circ$ were detected in the first CBCT. After the manual correction with the Solstice system, the pitch setup errors were reduced to $0.1-0.5^\circ$. This range may be caused by the adjustment step size of 0.25° we choose to use in our clinic for pitch correction and possible slight difference in location of the correction reference point of the CBCT and the center of the pitch-rotational axis of the head support.

Table 1: Setup accuracy for the 12 SRS patients treated using the Solstice system

	Lateral, mm	Longitudinal, mm	Vertical, mm	Pitch, °	Roll, °	Yaw, °
Mean	0.1	-0.2	0.2	0.2	0.2	0.6
Standard Deviation	0.3	0.3	0.4	0.5	0.6	0.9
Max	1	0.5	1	1.1	1	1.3